







Paramedic Donning Training Checklist

Suspected/Confirmed EVD Cases and/or Care Environment Coverall/Gown with Separate Hood and Foot Covers

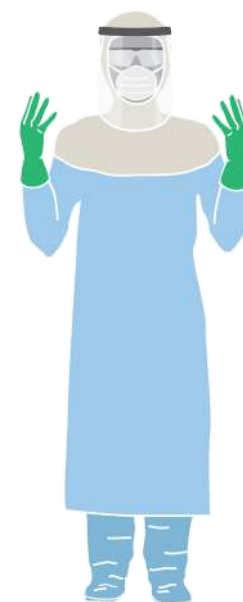

This checklist is designed to assist with training paramedics and first responders on the correct donning of personal protective equipment (PPE) for suspect/confirmed cases of EVD and their care environment. **Please Note: This is a sample checklist** that may need to be adapted to meet relevant standards of practice and/or specific manufacturer's user instructions for variations of approved PPE. The tool can be used to verify and document that all necessary donning steps are performed and in the proper sequence. Any errors or omissions identified during practice and skill proficiency demonstrations must be communicated immediately, to ensure specific instructions are provided and so appropriate corrective action are taken for the protection of the paramedic and/or first responder.

Before you begin, instruct paramedic(s) and/or first responder(s) to hydrate, tie back long hair and secure in place, and remove personal items such as hand and wrist jewelry. Gather and inspect PPE carefully. Ensure that the correct size is selected and that the PPE is in good sanitary and working condition. Damaged or defective PPE should not be used.

TRAIN.	PRAC.	PROF.	DONNING PPE PROCEDURE		COMMENTS (May include size of PPE)
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>		Perform hand hygiene: <ul style="list-style-type: none"> <input type="checkbox"/> Use alcohol-based hand rub (ABHR) or soap and water before putting on any PPE <input type="checkbox"/> Allow hands to dry completely before putting on any PPE 	
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>		Put on single-use (disposable) impermeable foot covers: <ul style="list-style-type: none"> <input type="checkbox"/> Select foot covers that extend to at least mid-thigh <input type="checkbox"/> Ensure foot covers allow for ease of movement <input type="checkbox"/> Adjust and verify for proper fit of PPE <input type="checkbox"/> Sit on clean chair, as needed 	
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>		Perform hand hygiene: <ul style="list-style-type: none"> <input type="checkbox"/> Use ABHR or soap and water <input type="checkbox"/> Allow hands to dry completely before putting on any PPE 	
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>		Put on inner gloves <ul style="list-style-type: none"> <input type="checkbox"/> Extend cuffs as far up arms as possible <input type="checkbox"/> Adjust and verify proper fit of PPE 	
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>		Put on single-use (disposable) impermeable coverall/gown: <ul style="list-style-type: none"> <input type="checkbox"/> Select coverall/gown large enough to allow unrestricted freedom of movement <input type="checkbox"/> Ensure cuffs of inner gloves remain tucked under sleeves <input type="checkbox"/> Seal opening of coverall and ensure no skin or clothing is exposed (if applicable) <input type="checkbox"/> Ensure a continuous barrier between foot covers and gown (if applicable) <input type="checkbox"/> Sit on clean chair, as needed 	

Paramedic Donning Training Checklist

Suspected/Confirmed EVD Cases and/or Care Environment
Coverall/Gown with Separate Hood and Foot Covers

TRAIN.	PRAC.	PROF.	DONNING PPE PROCEDURE	COMMENTS (May include size of PPE)
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Put on fit-tested N95 respirator: <ul style="list-style-type: none"> <input type="checkbox"/> Apply respirator as per manufacturer's user instructions <input type="checkbox"/> Fit flexible nose piece to bridge of nose <input type="checkbox"/> Perform seal check 	
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Put on eye protection (if applicable): <ul style="list-style-type: none"> <input type="checkbox"/> Apply safety eyewear as per manufacturer's user instructions 	
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Put on single-use (disposable) impermeable hood cover: <ul style="list-style-type: none"> <input type="checkbox"/> Ensure hood cover extends past shoulders and provides complete coverage of head, neck, and ears <input type="checkbox"/> Adjust and verify proper fit of PPE 	
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Put on face protection: <ul style="list-style-type: none"> <input type="checkbox"/> Apply full face shield as per manufacturer's user instructions <input type="checkbox"/> Ensure face shield provides coverage of entire front and sides of face; adjust to fit 	
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Put on outer gloves with extended cuffs: <ul style="list-style-type: none"> <input type="checkbox"/> Ensure gloves cover cuffs of coverall/gown and that no skin is exposed <input type="checkbox"/> Adjust and verify proper fit of PPE 	
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Verify donning PPE procedure: <ul style="list-style-type: none"> <input type="checkbox"/> Inspect PPE to ensure it is secure and full coverage has been achieved <input type="checkbox"/> Visually confirm sequence has been completed correctly 	

Name of Trainer: _____ Name of Worker: _____ Signature: _____ Date: _____